## Omnium1

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## Omnium1

### 3 in 1 adapter for D/A converter

With the Omnium1 - „3 in 1 Adapter“ for the existing D/A converter, you are able to connect all three applicators (full body mat, pad and spot) to the existing D/A converter of your Omnium1 system without having to unplug and reconnect anything.

To do so, please connect the Omnium1 - „3 in 1 adapter“ to the already existing D/A converter (side with two connection sockets).

Connect all applicators (and the OmniBrain System if available) to the Omnium1 - 3 in 1 adapter. The applicators may be screwed into any of the three available sockets. The application then automatically recognises the connected applicators.

**IMPORTANT:** Please ensure to download and install the latest version of the iMRsone software. Click (WIFI must be enabled) on „Settings -> About“ in the iMRsone app followed by clicking on the „Check App new version“ button.

Always restart the iMRsone application after connecting the 3 in 1 adapter! If the application is still active or loaded in the background, please close it and start it again. The Omnium1 „3 in 1 Adapter“ is only recognised by the system if it is connected to the existing D/A converter before the application is restarted.

Now select your desired applicator via the three applicator icons on the user interface of the iMRsone application and start the application as usual.

## Omnium1

### 3 in 1 adapter for D/A converter

With the Omnium1 - „3 in 1 Adapter“ for the existing D/A converter, you are able to connect all three applicators (full body mat, pad and spot) to the existing D/A converter of your Omnium1 system without having to unplug and reconnect anything.

To do so, please connect the Omnium1 - „3 in 1 adapter“ to the already existing D/A converter (side with two connection sockets).

Connect all applicators (and the OmniBrain System if available) to the Omnium1 - 3 in 1 adapter. The applicators may be screwed into any of the three available sockets. The application then automatically recognises the connected applicators.

**IMPORTANT:** Please ensure to download and install the latest version of the iMRsone software. Click (WIFI must be enabled) on „Settings -> About“ in the iMRsone app followed by clicking on the „Check App new version“ button.

Always restart the iMRsone application after connecting the 3 in 1 adapter! If the application is still active or loaded in the background, please close it and start it again. The Omnium1 „3 in 1 Adapter“ is only recognised by the system if it is connected to the existing D/A converter before the application is restarted.

Now select your desired applicator via the three applicator icons on the user interface of the iMRsone application and start the application as usual.